

Disclosures

2

4

- Successful completion: Participants should complete the entire program and submit required documentation. No partial credit will be given.
- 2. Conflict of interest: Employee of STERIS.
- Commercial company support: Fees are underwritten by education funding provided by STERIS.

1

Continuing Education

- STERIS Corporation is an approved provider of continuing nursing education by CBRN – provider # CEP 11681 and an approved Administrator Education Unit (AEU) and Infection Prevention Control (IPCH) provider by BASC – provider # 1417.
- This program is approved for:
- <u>0</u> hour(s) of GI Specific content credit by ABCGN (American Board of Certification for Gastroenterology Nurses),
- 1 AEU(s) & 1 IPCH(s) by BASC (Board of Ambulatory Surgery Certification), and
- 1_contact hour(s) of continuing education credit
- CBRN (California Board of Registered Nursing);
- CBSPD (Certified Board for Sterile Processing and Distribution); and
- HSPA (Healthcare Sterile Processing Association).

Continuing Education



Through a partnership with CCI*, it also meets CNOR* and CSSM* recertification requirements for perioperative nurses.

3


Learning Objectives


- 1. Define clean
- 2. List 4 factors that influence cleaning efficacy
- Describe the methods and equipment used to clean medical devices

What Is Cleaning?

5




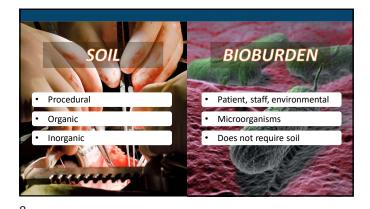


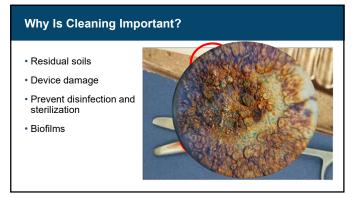

Biofilms · Fortified communities of microorganisms Resistant to disinfectants and sterilization Cleaning, in a timely fashion, is the most effective way to control biofilm


9 10

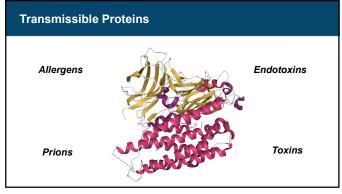

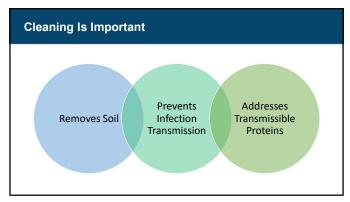

11 12




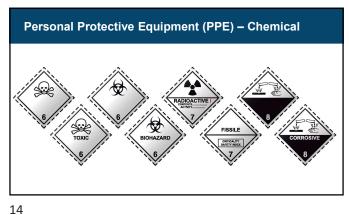






15

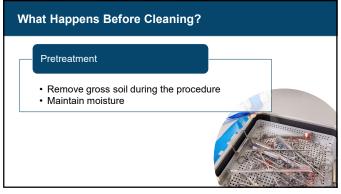





17




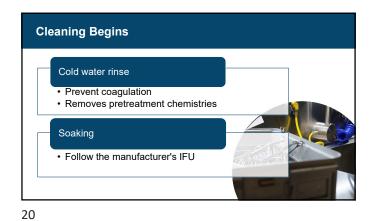




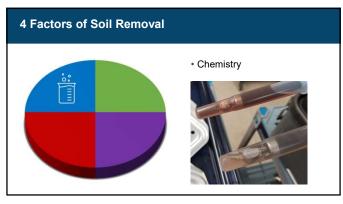


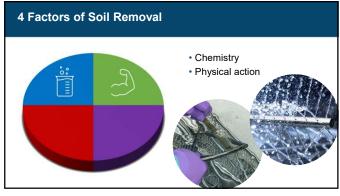

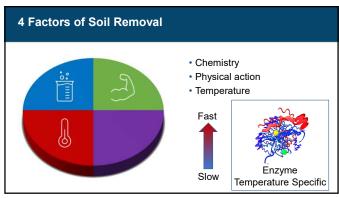


©January 2022 STERIS

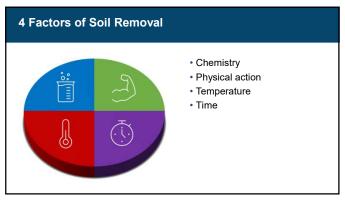

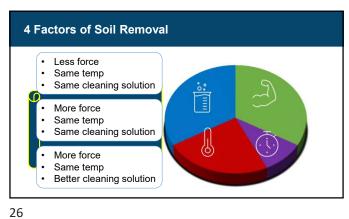






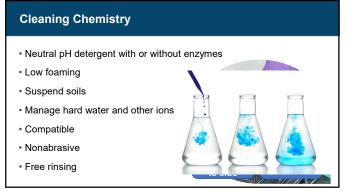







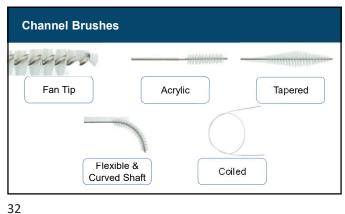





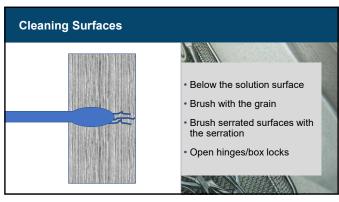




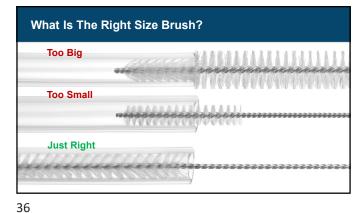

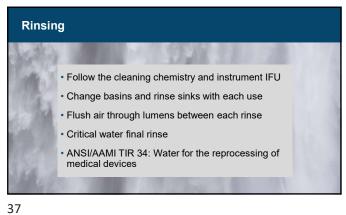

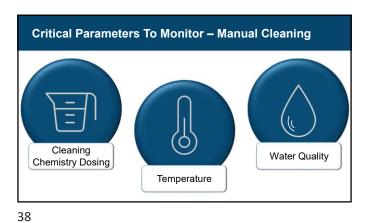





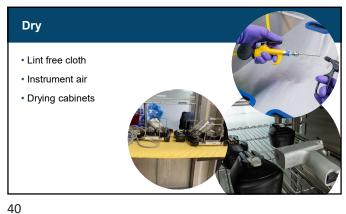







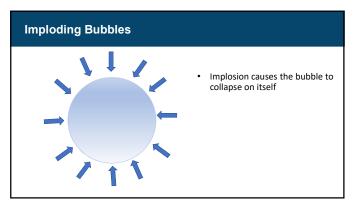


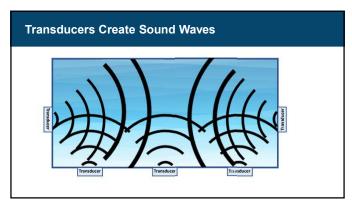

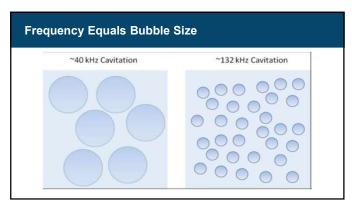

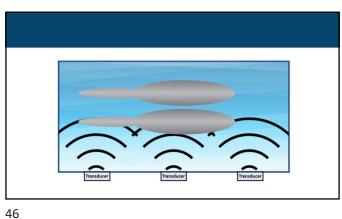

45

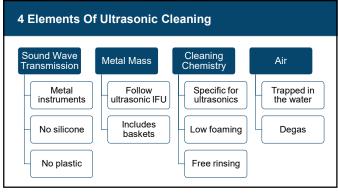




47





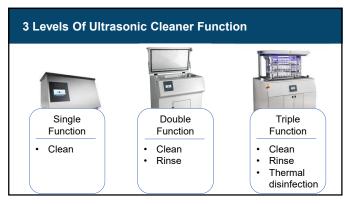








How Washer Disinfectors Clean Impingement Solution under pressure Spray arms • Irrigator connections

53 54




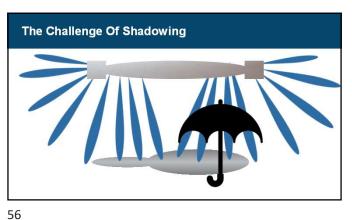

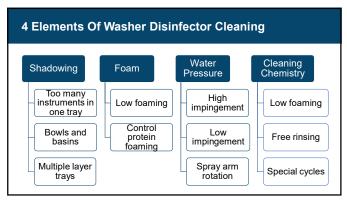

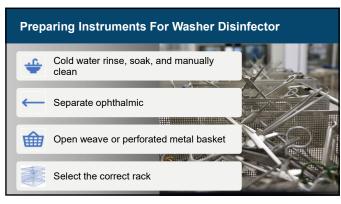



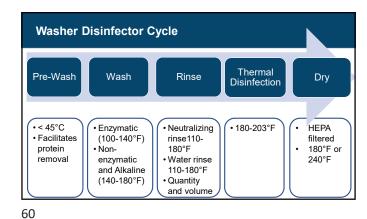








Quality Control

- Equipment calibration
- Equipment preventative and routine maintenance
- · Cleaning indicators
- Staff competency
- · Verification of the cleaning process

63 64

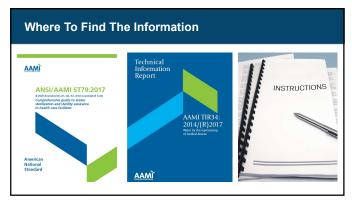

Action Items Review personal protective equipment Compare all manufacturer IFU against your current process Gauge your loading practices against best practices

65 66


References

- Alfa, M. J., & Nemes, R. (2004). Manual versus automated methods for cleaning reusable accessory devices used for minimally invasive surgical procedures. Journal of Hospital Infection, 58(1), 50-58.
- Association for the Advancement of Medical Instrumentation. (2017). ANSI TIR34:2014/(R)2017 Water for reprocessing of medical devices. Arlington, VA: Author
- Association for the Advancement of Medical Instrumentation. (2017). ANSI/AAMI ST79:2017 Comprehensive guide to steam sterilization and sterility assurance in health care facilities. Arlington, VA: Author
- International Association of Healthcare Central Service and Materiel Management (2016). Central Service Technical Manual. (8 edition). International Association of Healthcare Central Service and Materiel Management

67 68





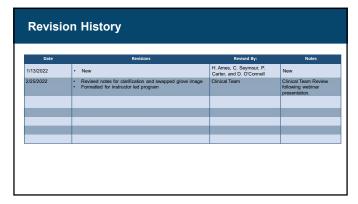

69 70