

Delivering a Meal Bolus with an Insulin Pump

**** Always make sure the Bolus Amount is actually delivered by the pump.**

Minimed Paradigm - Bolus Wizard

- Hit "B" button -
- Use "Up" or "Down" arrow to enter Blood glucose
- Hit "Act"
- Use "Up" arrow to enter grams carbohydrate - (You can deduct grams for activity at this point, if necessary)
- Hit "Act" - Review Estimate Details, you can scroll down using arrows.
- Hit "Act" to deliver bolus

www.minimed.com

Minimed 630G or 670G - Bolus Wizard

Menu of Highlighted word will open when center (round) button is pressed.

- Hit round (center) button to go to "Bolus"
- Select "Bolus Wizard" to open that screen
- "BG" is highlighted, select with round button
- Use "up" or "down" arrows to enter blood glucose
- Hit round button to move to "Carbs," select, use "up" arrow to enter carb amount
- Hit round button to go to Bolus Wizard calculation, hit round button to deliver bolus
- If calculation is incorrect, use back arrow to correct.

Animas - ezCarb

- Hit "OK" to wake up screen
- Hit "OK" on "Menu"
- Hit "OK" on Bolus
- Use Down arrow to "ezCarb" - Hit "OK"
- Use Up arrow to enter grams carb (You can deduct grams for activity, if necessary)
- - Hit "OK"
- "Add BG" - Hit "OK"
- Use Up or Down arrow to enter BG
- Hit "OK" on "Show Result"
- Use Arrow to match "Total" - Hit "OK"
- Hit "OK" on "GO"

www.animas.com

OmniPod

- Check blood glucose using PDM.
- When blood glucose reading appears, press "Next" to continue.
- Press "Yes" for "Are you eating now?"
- Use Up/Down arrows to enter carb grams (You can deduct grams for activity, if necessary)
- Press "?" to view bolus calculation-then "Close"
- Press "Enter" to deliver bolus.

www.myomnipod.com

t:slim

- Tap button on top of pump to open
- Tap circles "1," "2," "3"
- Tap "Bolus"
- Tap "0 grams"
- Enter number of carb units, then "Done"
- Tap "Add BG"
- Enter Blood glucose, then "Done"
- You can "View Calculation" or tap "Next"
- "Confirm Request?" Tap "Yes" (or "No" to go back to previous screen
- Tap "Deliver" to deliver bolus, and pump will say "bolus Initiated."

www.tandemdiabetes.com/